

# VaccinePorter<sup>®</sup> 16




## Prior to use:

Before inserting each Medicool<sup>®</sup> into the VaccinePorter<sup>®</sup> system, please follow these instructions:

- Refrigerate 14 x Medicool<sup>®</sup> 28 at +5°C, ± 3°C for a **minimum** of 24 hours\*.
- Medicool<sup>®</sup> cool packs should be stacked flat on storage shelves no more than 3 high, with 50mm of free air circulation around the packs.

## Instructions for Use - Please Turn Over

\*Due to the potential variation in fridge performance, frequency of opening and other fridge contents, it is recommended that Medicool<sup>®</sup> pre-conditioning is validated by the user.

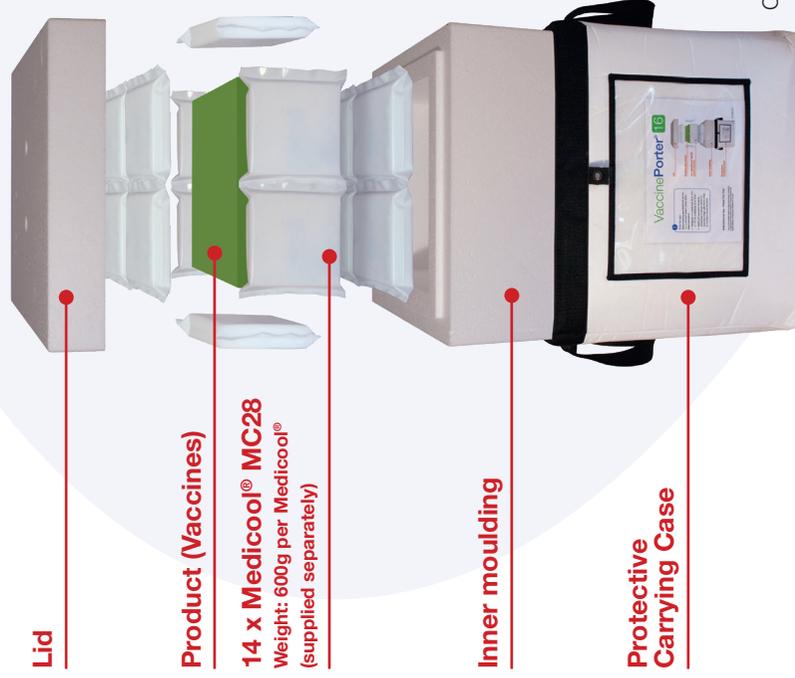

CP1423/1017

## Instructions for Use

### System assembly and product packing

Pack and seal the VaccinePorter<sup>®</sup> system swiftly, as follows:

1. Place 4 x Medicool<sup>®</sup> 28 on the base of the system.
2. Place the product to be transported in the centre of the VaccinePorter<sup>®</sup>.
3. Place 2 x Medicool<sup>®</sup> 28 on each side of the system, and one on each end, surrounding the product load.
4. Place 4 x Medicool<sup>®</sup> 28 on top of the product.
5. Close the lid and ensure it is sealed by closing the outer carrying case immediately.
6. Only open the VaccinePorter<sup>®</sup> if necessary and remember to re-seal as soon as possible.

Although the VaccinePorter<sup>®</sup> has been validated so that the product can be added and removed from the sealed system during use, do not open unnecessarily and remember to re-seal the system quickly.

### To re-use the system:

In case you decide to re-use the system, please follow these steps below prior to conditioning the Medicool<sup>®</sup> cool packs:

- A weight check and visual squeeze check will need to be carried out to ensure there is no loss of liquid. If the Medicool<sup>®</sup> is ±20g different to the weight shown on the Medicool<sup>®</sup> (overleaf) or shows any signs of leaking, then it is recommended that it is disposed of.
- We recommend that any chipped or broken polystyrene inners are replaced before use as this can affect the thermal performance of the VaccinePorter<sup>®</sup>.

Further information is available from:



Helapet Limited, Lister House, Blackburn Road, Houghton Regis, Bedfordshire. LU5 5BQ  
Telephone 0800 0328 428 Fax +44 (0) 1582 501981

E-mail: sales@helapet.co.uk | Web: helapet.co.uk